

# Upholstery Installation-Set up Instructions /037S Dental Chair

2-8-05

*Time Required: Approximately 20 minutes*

## Tools Required:

Phillips Head Screwdriver


11/16" Socket Wrench

6mm Hex Key Wrench


Small Adjustable Wrench



1. Remove tagged red shipping bolt with 11/16" socket wrench.

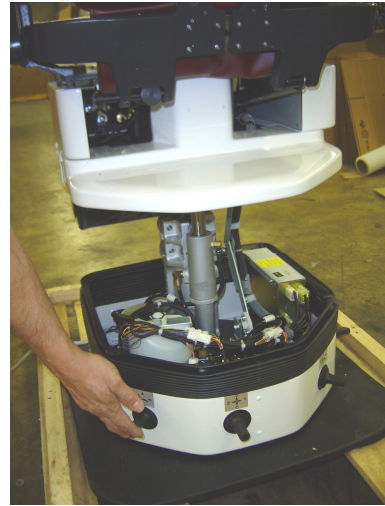

2. Plug chair power cord into an electrical outlet and raise chair base.

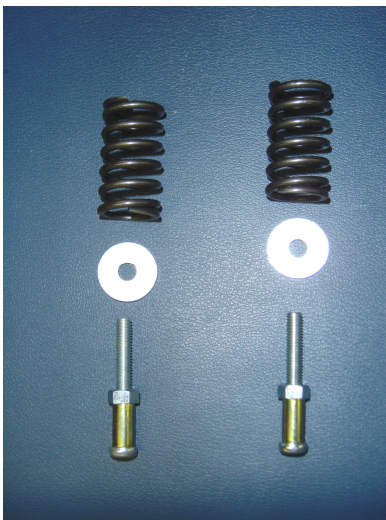

3. Arm rest screws/  
washers/  
compression springs



4. Insert armrest into the top of the armrest bracket.  
Insert washer and spring over screw, then insert into the bottom of the armrest bracket.



5. Tighten screw with a Phillips head screwdriver.